



OVERVIEW
**ONE-PIECE
IMPLANTS**



ONE-PIECE IMPLANTS



 **ONE-PIECE PLATFORM**

*Simple solution to
complex problems*

**ONE-STAGE
IMPLANTATION
ALREADY EXISTS
AND IT'S WORKING!**

*“The one-stage concept allows to solve problems
simply, quickly, effectively, economically”*

Dr. Georgi Pataraya

Implantologist, Craniofacial Surgeon

P9S

DENTAL IMPLANT



**One-piece
Platform**

One-Piece implant, with 4 restoration options in Snap, Cemented, Screw retained, and a bendable neck (up to 20°), Compressively threaded implant, good for all bone types.



Ø2.4mm	Ø3.0mm	Ø3.2mm	Ø3.5mm	Ø4.2mm	Ø5.0mm
-	-	-	-	-	6 mm
-	8 mm	8 mm	8 mm	8 mm	8 mm
10 mm	10 mm	10 mm	10 mm	10 mm	10 mm
11.5 mm	11.5 mm	11.5 mm	11.5 mm	11.5 mm	11.5 mm
13 mm	13 mm	13 mm	13 mm	13 mm	13 mm
16 mm	16 mm	16 mm	16 mm	16 mm	16 mm

BASIC SURGICAL KIT //

DRILLS

1.5 2.0 2.5 2.8 3.2 3.7 4.0 4.5 5.5

DRIVER **DRIVER "D"** **DRIVER "N"**

K5 - 9 K5 - 15 K5D - 9 K5D - 15 K5N - 9 K5N - 15

K8 - 23 K8 - 28 K8D - 23 K8D - 28 K8N - 23 K8N - 28

PROSTHETICS

K1 - 9 K1 - 15 K2 - 9 K2 - 15 K9 - 23 K7 K7N

Parallel pin

SGS Dental

DR 1.5 Ref. num.: C9215	DR 2.0 Ref. num.: C992	DRCS 2.5 Ref. num.: C8025	DRCS 2.8 Ref. num.: C8028	DRCS 3.2 Ref. num.: C8032
DRCS 3.7 Ref. num.: C8037	DRCS 4.0 Ref. num.: C8040	DRCS 4.5 Ref. num.: C8045	DRCS 5.5 Ref. num.: C8055	

// B // CHOICE

with
**Spiral
Drill kit**

Surgical drills
recommended for
bone type D1-D2.









K5 - 9 Driver - B59	K8D - 23 Driver - DB823	K1 - 9 Driver - B19	K7N Retrieving screw - NB7
K5 - 15 Driver - B515	K8D - 28 Driver - DB828	K1 - 15 Driver - B115	Parallel pin C8920 / 8927
K8 - 23 Driver - B823	K5N - 9 Driver - NB59	K2 - 9 Driver - B29	K5 - 0 Driver - B50
K8 - 28 Driver - B828	K5N - 15 Driver - NB515	K2 - 15 Driver - B215	RBT Ratchet - C88
K5D - 9 Driver - DB59	K8N - 23 Driver - NB823	K9 - 23 Adaptor - B923	D9 Handle - C87
K5D - 15 Driver - DB515	K8N - 28 Driver - NB828	K7 Retrieving screw - B7	



ACCESSORIES

Healing cap	Analog	Transfers		Temporary		Immediate Loading/Final restoration	
		Open tray	Closed tray	Click-on		Screw retained	
H-S4	A1-S4	T1-S4	TP-S4	S4a - Plastic	S4a - Peek	T-S4	P-S4
							

INSTRUMENTS


Key	Drivers		Expanders				
	9/15mm	23/28mm	∅ 3.0	∅ 3.2	∅ 3.5	∅ 4.2	∅ 5.0
K 6	K 5 D	K 8 D	K 13	K 13	K 13	K 13	K 13
							




Implantation Options




Can be used with a Motor via a K8 and K8D

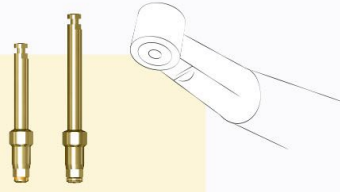


Can be used with a ratchet or Torque ratchet via a K5 or a K5D or even a K5-0

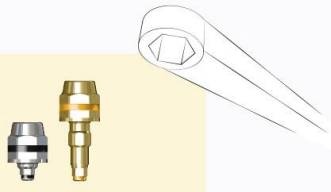

Implantation Process I




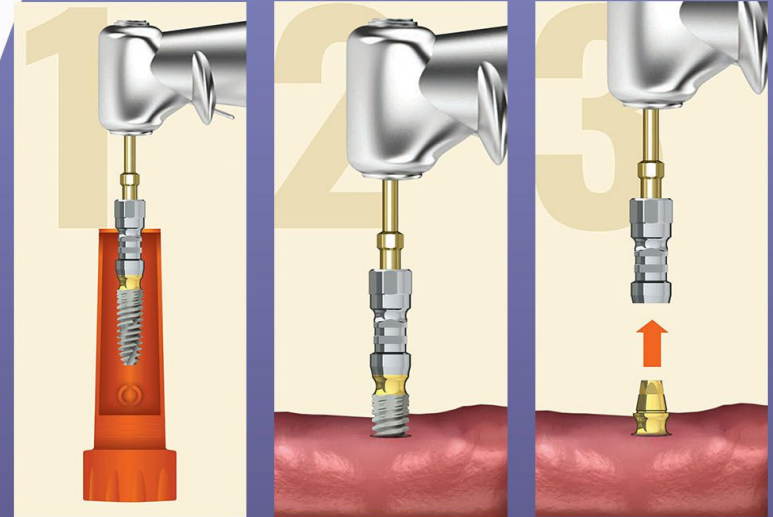


Implantation Process II

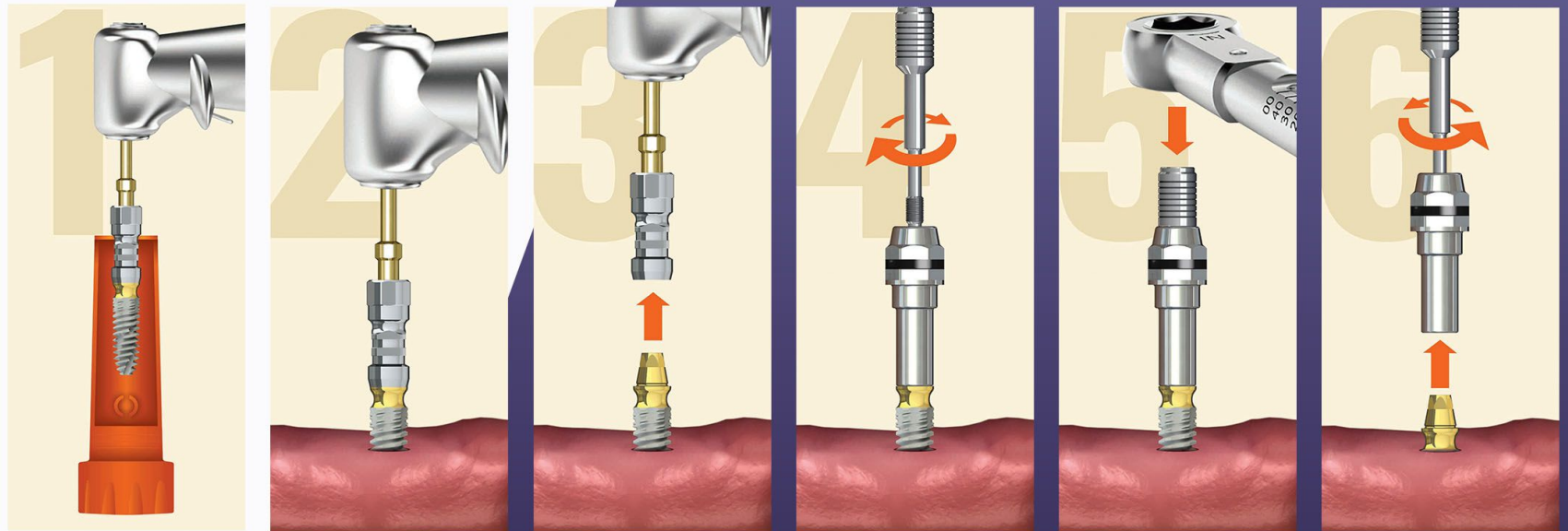




Optional bending Procedure

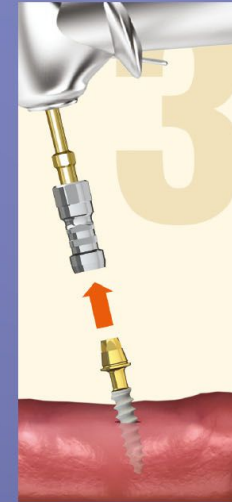
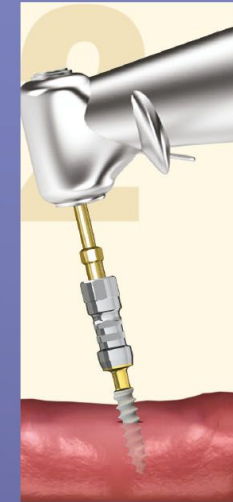
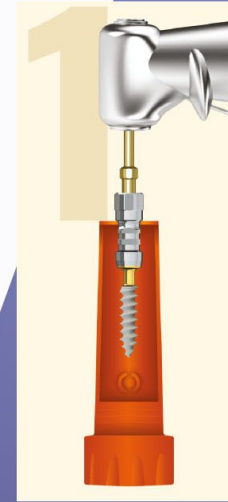
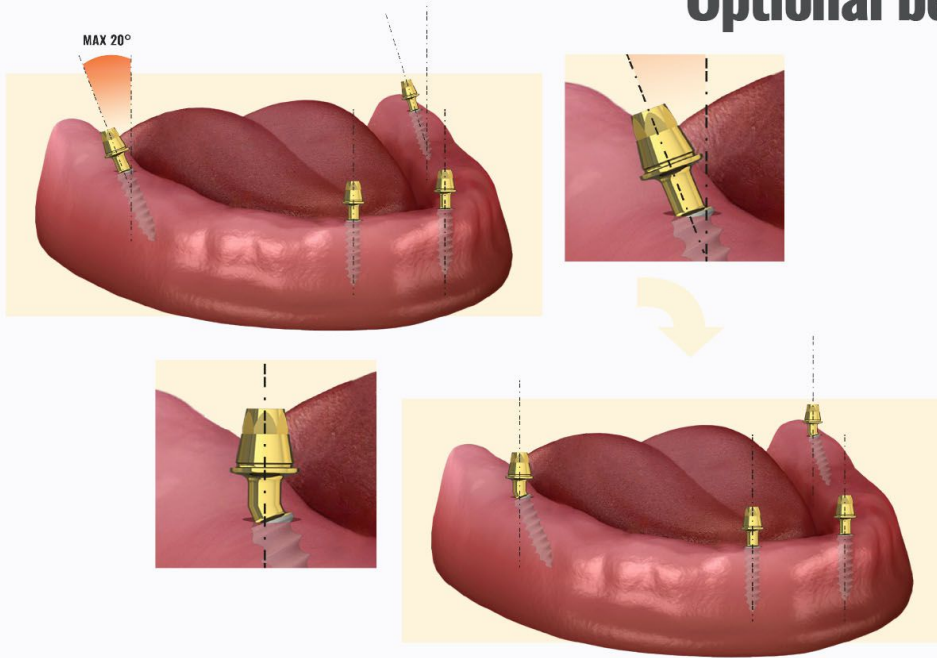


▲ The outreach of the ratchet can be used to apply the required pressure for the bending of the Implant Neck, please apply the pressure slowly and keep in mind the neck can only be bent once!
A repeated attempt to bend the neck back and forth will significantly weaken the implant and may lead to a possible failure.

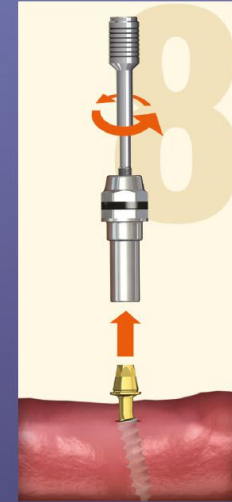





Immediate Loading

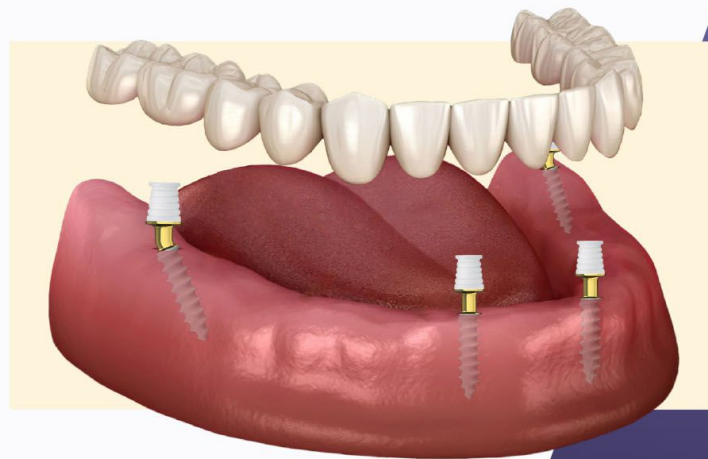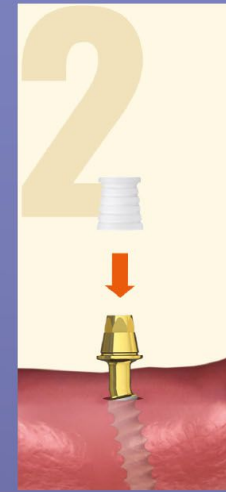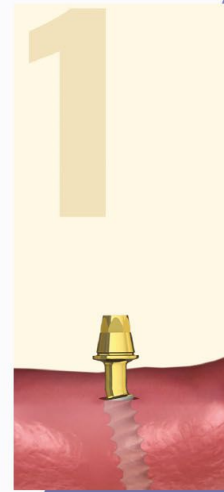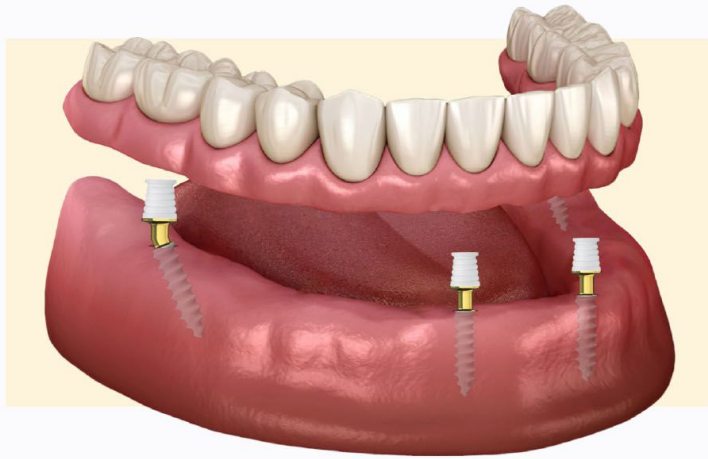




Immediate Loading

Peek Overdenture

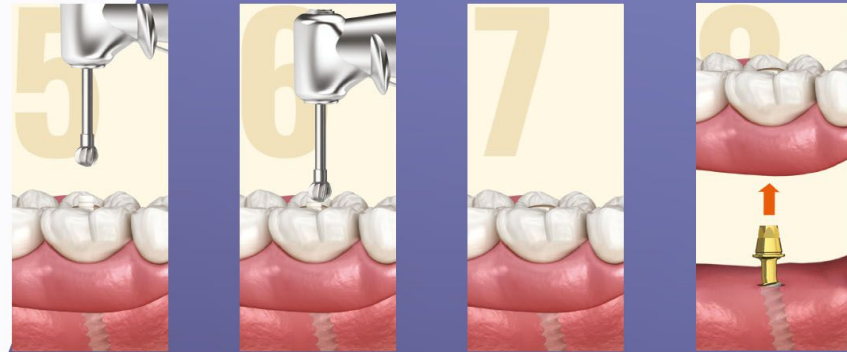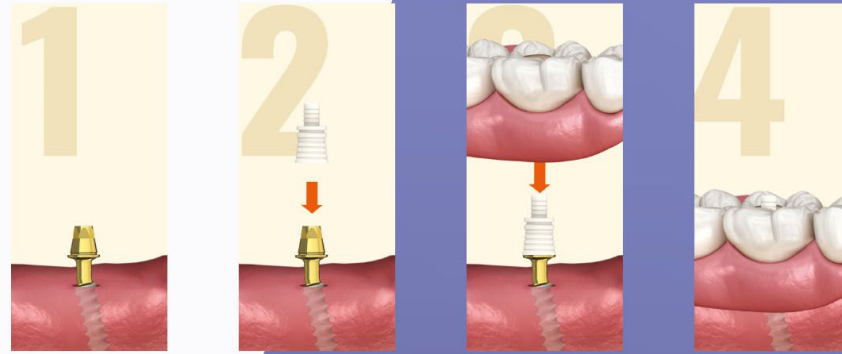

Peek Overdenture Removal





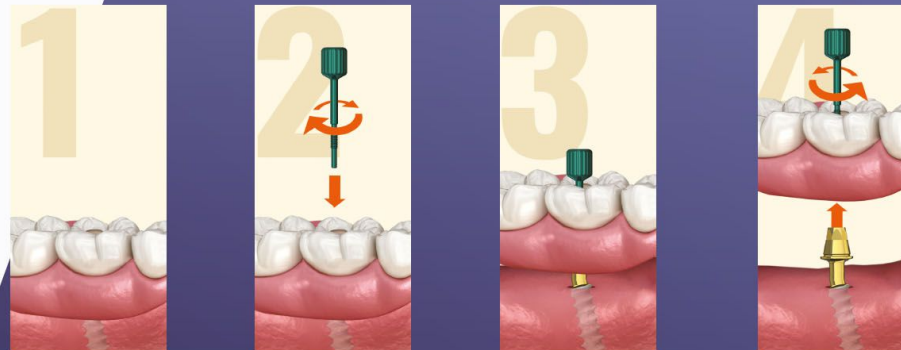




Immediate Loading







Screw Retained
Overdenture

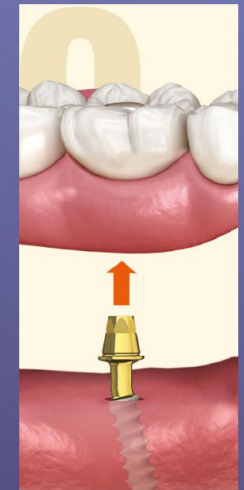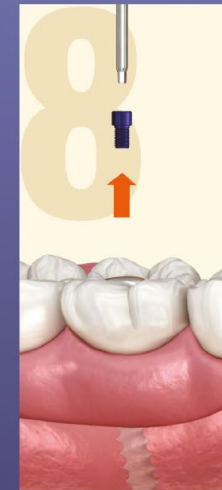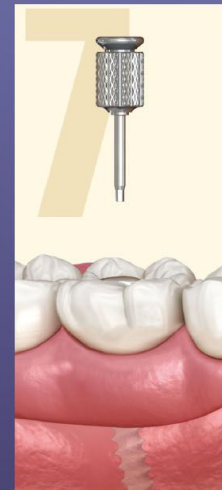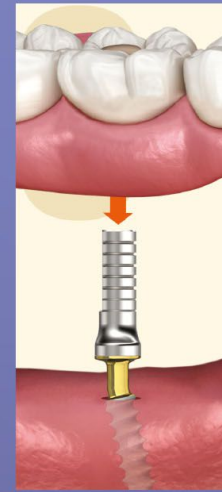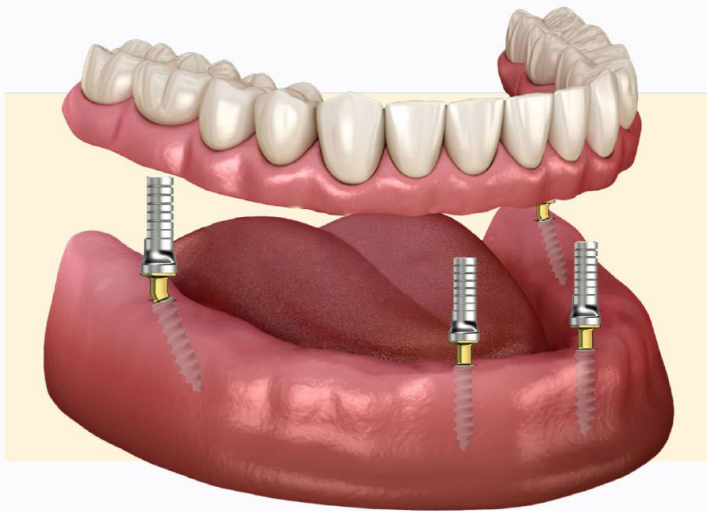





Impression Copying

For Closed Tray

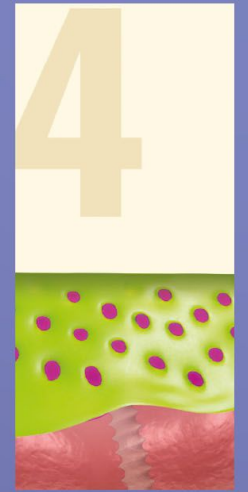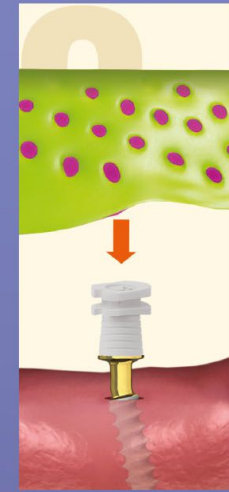
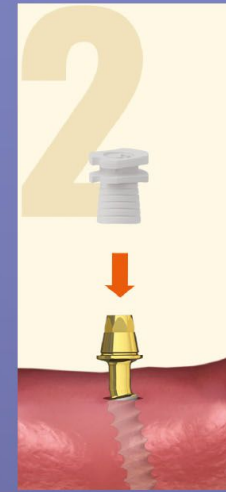
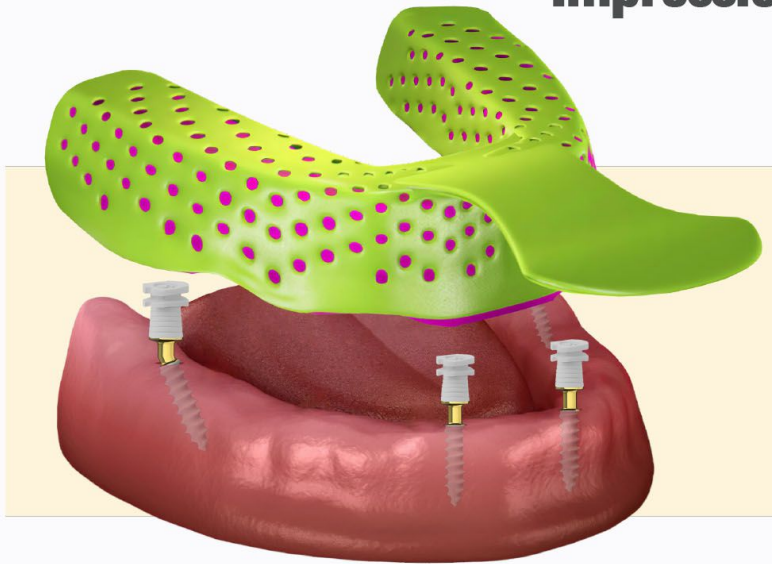


For Open Tray

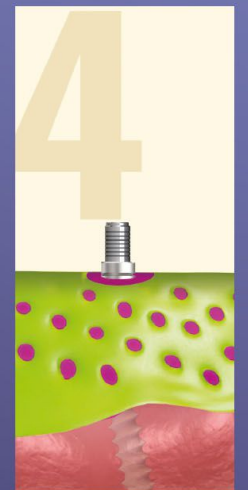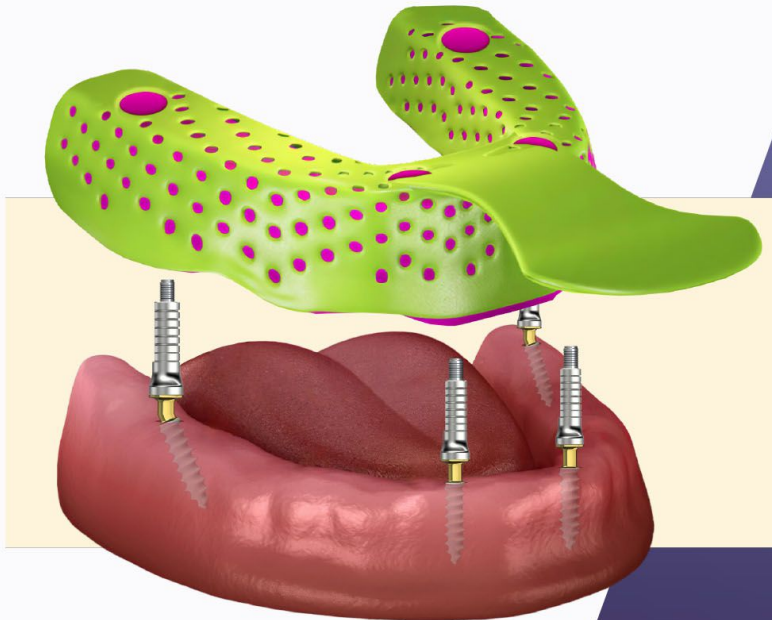



Clinical Case

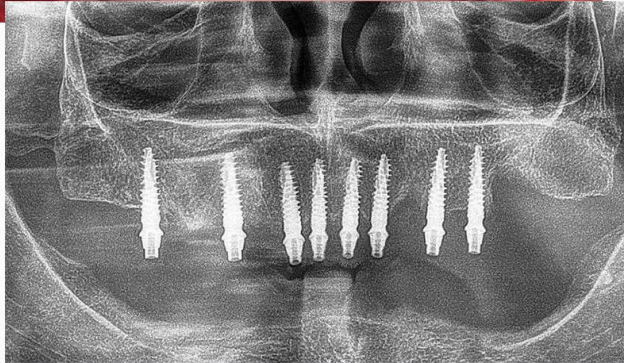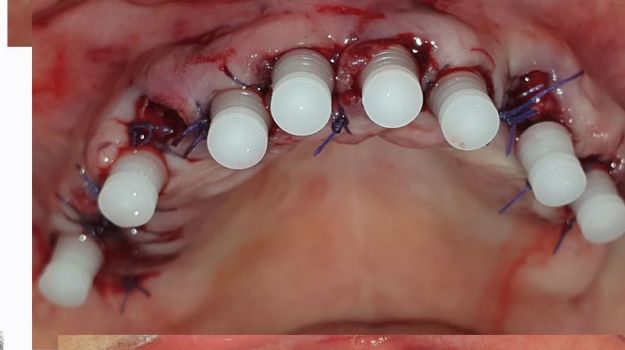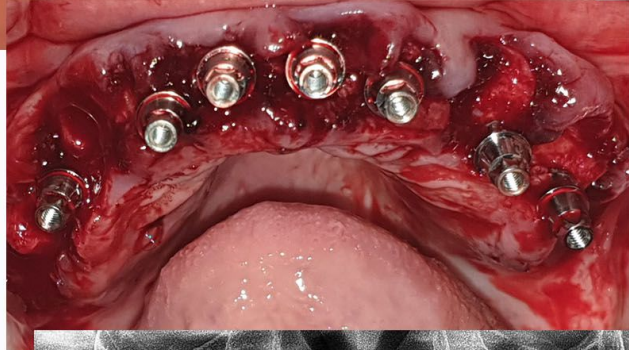

**SIMPLICITY
GUARANTEES
SUCCESS**

SGS Dental Implant System Holding - Zn
Talhofstrasse 9, CH-9000 St. Gallen, **Switzerland**
www.sgs-dental.com

FL-9494, Schaan, Landstrasse 27
Liechtenstein
Tel.: 00423 233 5050, 00423 233 5051
Fax: 00423 233 5052

SGS Dental Georgia Ltd. Worldwide Logistic Center
0162, 39, I. Chavchavadze Ave.
Tbilisi, Georgia
Tel: +995 597 970801
Email: georgia@sgs-dental.com
www.sgs-dental.ge

